




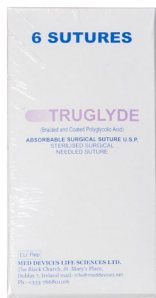

THD SLIDE KIT

THD SUTURE	THD SLIDE	THD DOPPLER PROBE
		

Use

THD Slide Kit provides all instruments needed to perform the THD procedure; treatment of varying grades of haemorrhoidal disease.

General description

THD Slide is specifically designed for immediate use with the THD Revolution/THD Evolution doppler dearterialiser. THD Slide anoscope/proctoscope is specifically designed for improving the THD procedure while treating the higher grades of the haemorrhoidal disease. The specific shape of the anoscope/proctoscope, which includes a port for the THD Doppler Probe, allows ligation and prolapse reduction with a single tool and reduces procedure time. The THD Slide anoscope/proctoscope is made of clear, medical grade, plastic which creates, in tandem with the use of the fibre optic tip, ideal lightning of the operative field. The THD Slide anoscope/proctoscope is designed to maintain the correct tension of the anal cavity during the procedure and presents placement for the THD needle holder. The supplied doppler probe lodges into the anoscope/proctoscope and is thereby positioned with the anal mucosa for an accurate detection of the haemorrhoidal arteries. The box also contains 2 re-usable, 5 shots, Doppler probes, 10 sutures packs with 6 sutures each and 10 Anal Spon (haemostatic anal sponges).



Dimensional Features	
THD SLIDE	
<p>Technical drawing of the THD Slide. The side view shows a handle with a length of 105 mm and a total length of 172 mm. The top view shows a width of 10 mm and a height of 6 mm. The cross-section view (SEZIONE B-B) shows an internal dimension of 27x19 mm and an external dimension of 33x23 mm. The operational length is 100 mm. The operative windows are 30x10 mm minimum. The total length is 172 mm. The handle length is 105 mm.</p>	
Internal Dimension	27x19 mm (1.063x0.748 in)
External Dimension	33x23 mm (1.299x0.905 in)
Operational Length	100 mm (3.937 in)
Operative Windows (Variable)	30x10 mm (1.181x0.393 in) minimum
Total Length	172 mm (6.772 in)
Handle length	105 mm (4.133 in)

		THD Slide		
P/N	Q.ty	Description	Risk Class	EMDN
800066-10	10	THD Slide	-	-
include 710004	10	THD Slide (sterile)	Ila	Z12020604
include 800056	2	Doppler Probes with 5 activations each	Ila	Z12059005
include 880000	10	n. 6 THD Suture (sterile)	III	H0101010201
include 800050	10	Scrofolon Anal (sterile)	III	M040503

Associated Equipment		
P/N	Qty	Description
800133	1	THD REVOLUTION (generator for doppler dearterialiser)