




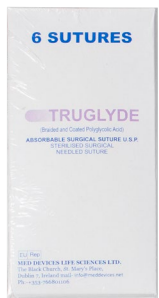

# THD SLIDE ONE KIT

<b>THD SUTURE</b>	<b>THD SLIDE</b>	<b>THD DOPPLER PROBE</b>
		

## Use

THD Slide One Kit provides all instruments needed to perform the THD procedure; treatment of varying grades of haemorrhoidal disease.

## General description

THD Slide is specifically designed for immediate use with the THD Revolution/THD Evolution doppler dearterialiser. THD Slide anoscope/proctoscope is specifically designed for improving the THD procedure while treating the higher grades of the haemorrhoidal disease. The specific shape of the anoscope/proctoscope, which includes a port for the THD Doppler Probe, allows ligation and prolapse reduction with a single tool and reduces procedure time. The THD Slide anoscope/proctoscope is made of clear, medical grade, plastic which creates, in tandem with the use of the fibre optic tip, ideal lightning of the operative field. The THD Slide anoscope/proctoscope is designed to maintain the correct tension of the anal cavity during the procedure and presents placement for the THD needle holder. The supplied doppler probe lodges into the anoscope/proctoscope and is thereby positioned with the anal mucosa for an accurate detection of the haemorrhoidal arteries. The THD Slide One box contains 10 individually packed disposable Doppler probes, which can be immediately be used without preventive cleaning and sterilisation. The box contains 10 adsorbable sutures packs (6 sutures each pack) and 10 Anal Spon haemostatic sponges.



## Product Datasheet

P/N 800065 - 1  
P/N 800065 - 5  
P/N 800065 - 10

EN

Dimensional Features	
THD SLIDE	
Internal Dimension	27x19 mm (1.063x0.748 in)
External Dimension	33x23 mm (1.299x0.905 in)
Operational Length	100 mm (3.937 in)
Operative Windows (Variable)	30x10 mm (1.181x0.393 in) minimum
Total Length	172 mm (6.772 in)
Handle length	105 mm (4.133 in)

THD Slide One (Sterile)			
P/N	Q.ty	Description	Risk Class
800065-10	10	THD Slide One Kit (sterile)	
include 710004	10	THD Slide (sterile) – with 1 proctoscope, 1 needle holder, 1 pusher	IIa
include 800063	10	Disposable Doppler Probes (sterile)	IIa
include 800050	10	Scrofolon Anal (sterile)	III
include 880000	10	n. 6 THD Suture (sterile)*	III
or			
include 880000	10	n. 6 THD Suture (sterile)**	III

\* KIT including Truglyde sutures

\*\* KIT including Assufil sutures

THD Slide One (Sterile)		
P/N	Q.ty	Description
800065-1	1	THD SLIDE ONE KIT
800065-5	5	THD SLIDE ONE KIT
800065-10	10	THD SLIDE ONE KIT

THD S.p.A. Via per Carpi, 15/B 42015 Correggio (RE), Italy – Tel +39 0522 634311 – [www.thdlab.com](http://www.thdlab.com)

Administrative Offices Via per Carpi 26/B, 42015 Correggio (RE), Italy  
Reg. Imp., Cod. Fisc. & VAT IT02111430357 – Capitale Sociale € 4.000.000,00 I.V.

THD360025\_EN\_Rev02\_20250218\_THD Slide One



## Product Datasheet

P/N 800065 - 1

P/N 800065 - 5

P/N 800065 - 10

EN

Associated Equipment		
P/N	Qty	Description
800133	1	THD REVOLUTION
800000	1	THD EVOLUTION

THD360025\_EN\_Rev02\_20250218\_THD Slide One

THD S.p.A. Via per Carpi, 15/B 42015 Correggio (RE), Italy – Tel +39 0522 634311 – [www.thdlab.com](http://www.thdlab.com)

Administrative Offices Via per Carpi 26/B, 42015 Correggio (RE), Italy  
Reg. Imp., Cod. Fisc. & VAT IT02111430357 – Capitale Sociale € 4.000.000,00 I.V.