





THD SLIDE ENV (LIGHT)

THD SLIDE ENV	THD DOPPLER PROBE
	


Use

THD Slide Kit provides all instruments needed to perform the THD procedure; treatment of varying grades of haemorrhoidal disease.

General description

THD Slide is specifically designed for immediate use with the THD Revolution/THD Evolution doppler dearterialiser. THD Slide anoscope/proctoscope is specifically designed for improving the THD procedure while treating the higher grades of the haemorrhoidal disease. The specific shape of the anoscope/proctoscope, which includes a port for the THD Doppler Probe, allows ligation and prolapse reduction with a single tool and reduces procedure time. The THD Slide anoscope/proctoscope is made of clear, medical grade, plastic which creates, in tandem with the use of the fibre optic tip, ideal lightning of the operative field. The THD Slide anoscope/proctoscope is designed to maintain the correct tension of the anal cavity during the procedure and presents placement for the THD needle holder. The supplied doppler probe lodges into the anoscope/proctoscope and is thereby positioned with the anal mucosa for an accurate detection of the haemorrhoidal arteries. The box also contains 1 re-usable, 10 shots, Doppler probes and 1 THD needle holder re-usable.

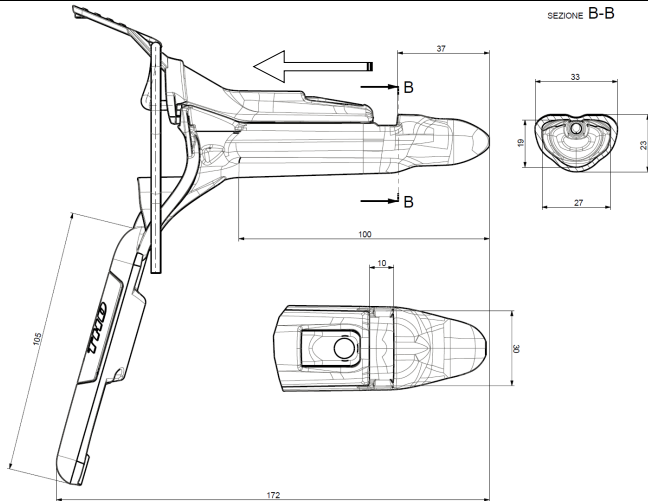

Dimensional Features	
THD SLIDE ENV (LIGHT)	
 <p>Technical drawing of the THD Slide Env (Light) showing side, top, and cross-section views with dimensions. The side view shows a handle length of 105 mm and a total length of 172 mm. The top view shows a width of 33 mm and a depth of 23 mm. The cross-section view shows a width of 33 mm and a depth of 23 mm. The operational length is 100 mm. The internal dimension is 27x19 mm. The external dimension is 33x23 mm. The operative windows are 30x10 mm minimum. The total length is 172 mm. The handle length is 105 mm.</p>	
Internal Dimension	27x19 mm (1.063x0.748 in)
External Dimension	33x23 mm (1.299x0.905 in)
Operational Length	100 mm (3.937 in)
Operative Windows (Variable)	30x10 mm (1.181x0.393 in) minimum
Total Length	172 mm (6.772 in)
Handle length	105 mm (4.133 in)

THD Slide Env (Light)		
P/N	Q.ty	Description
800169-10	10	THD Slide Env (Light)
include 710009	10	THD Slide Env (sterile) in a bag – with 1 proctoscope, 1 pusher
include 700001	1	Doppler Probes with 10 activations each
include 750001	1	THD needle holder re-usable

Associated Equipment		
P/N	Qty	Description
800133	1	THD REVOLUTION (generator for doppler dearterialiser)
800000	1	THD EVOLUTION (generator for Doppler dearterialiser – previous model)