




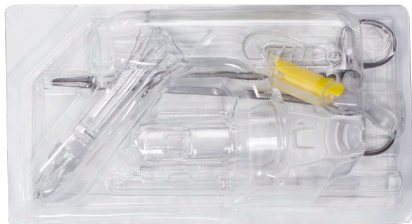


# THD SLIDE ONE FAST

<b>THD SLIDE</b>	<b>THD DOPPLER PROBE</b>	<b>THD SUTURE</b>	<b>THD BLOCK</b>
			

## Use

THD Slide One Fast is a kit specifically designed for immediate use with the THD Revolution doppler dearterialiser.

## General description

THD Slide anoscope/proctoscope is specifically designed for improving the THD procedure while treating the higher grades of the haemorrhoidal disease. The particular shape of the Slide anoscope/proctoscope, with the presence of a sliding carriage, makes it possible to carry out ligation and prolapse resolution with a single instrument, thus causing less discomfort to the patient and speeding up the procedure. The THD Slide anoscope/proctoscope is made of transparent, medical grade, plastic which allow, together with the use a specially shaped fiber optic tip, ideal lightning of the operating field. The THD Slide anoscope/proctoscope is designed to maintain the correct tension of the anal cavity during the procedure and a special slot for the THD needle holder. The supplied doppler probe lodges into the anoscope/proctoscope and it is in direct contact with the anal mucosa for an accurate detection of the haemorrhoidal arteries.

The THD Slide One Fast box contains:

- 10 THD Slide disposable kits in sterile blister packs, for immediate use
- 10 individually packed disposable Doppler probes, which can be immediately used without preventive cleaning and sterilisation
- 10 adsorbable sutures packs (6 sutures each pack), needle 5/8 – 26.5 mm
- 10 THD Block packs (surgical, absorbable sutures with self-locking system, unidirectional with final lock system) with 3 sutures each, needle 1/2 circle – 30 mm
- 10 Anal Spon haemostatic sponges.



Dimensional Features	
THD SLIDE	
Internal Dimension	27x19 mm (1.063x0.748 in)
External Dimension	33x23 mm (1.299x0.905 in)
Operational Length	100 mm (3.937 in)
Operative Windows (Variable)	30x10 mm (1.181x0.393 in) minimum
Total Length	172 mm (6.772 in)
Handle length	105 mm (4.133 in)

THD Slide One Fast (Sterile)				
P/N	Q.ty	Description	Risk Class	EMDN
800280-10	10	THD Slide One Fast (sterile)	-	Z12020604
includes	710004	10 THD Slide (sterile) – with 1 proctoscope, 1 needle holder, 1 pusher	Ila	Z12020604
includes	800063	10 Disposable Doppler Probes (sterile)	Ila	Z12059005
includes	800050	10 Scrofolon Anal (sterile)	III	H010101010301
includes	880059	10 THD Block (3 sutures - sterile)	III	M040503
includes	880000	10 n. 6 THD Sutures (sterile)*	III	H0101010201
or				
includes	880000	10 n. 6 THD Sutures (sterile)**	III	H0101010201

\*KIT including sutures Truglyde

\*\*KIT including sutures Assufil

Associated Equipment		
P/N	Qty	Description
800133	1	THD REVOLUTION